USER MANUAL

Partial Sample Sealing Kit Version Sydney

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Tel. +41 71 982 88 11
info.special@berlinger.com
www.berlinger.com

TABLE OF CONTENTS

WHAT IS INCLUDED	2
INTENDED USE	2
LIMITATIONS OF THE PRODUCT	2
HANDLING INSTRUCTION	3
WARNINGS AND PRECAUTIONS	4
ADDITIONAL HELPFUL MATERIALS	4

WHAT IS INCLUDED

In this **Partial Sample Sealing Kit Version Sydney** kit, you will find:

- One (1) uniquely numbered tamper-evident void tape (blue)
- One (1) white plastic stopper

INTENDED USE

Partial Sample Sealing Kit Version Sydney is intended to be used to temporarily store urine samples for the purpose of anti-doping and drugs of abuse testing outside of a medical setting and not on order of a health care professional.

This kit is not intended to be used for or in assistance of disease screening or diagnosis. This kit is intended to be used in accordance with the current international standard on testing and investigations established by the World Anti-doping Agency (WADA) and samples used with this kit should be sent to, processed by, and interpreted by WADA accredited laboratories.

The Partial Sample Version Sydney must only be used for BREG-KIT Urine Sample Container **in a styrofoam box**.

LIMITATIONS OF THE PRODUCT

- Optimal performance of this kit requires appropriate specimen collection, handling, preparation, and storage.
- This product is not intended to be used for or in assistance of disease screening or diagnosis.

HANDLING INSTRUCTION

PREPARATION

Please make sure that the content of the Partial Sample Sealing Kit Version Sydney is complete.

HANDLING THE PARTIAL SAMPLE SEALING KIT VERSION SYDNEY

1. Fill the collected urine into a BEREK-Kit Urine Sample Container bottle in accordance with WADA ISTI and close the partially filled bottle with the white plastic stopper.
2. Leave the red ring on the bottle neck.
3. Place the security cap back on top of the bottle. Do not screw the cap!
4. The second bottle remains unopened.
5. Put all components into the styrofoam box and close it.
6. Seal the styrofoam box with the uniquely numbered tamper-evident void tape. The BEREK-Kit is now temporarily sealed.

NOTES

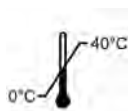
- Use the white strap to remove the backing paper from the uniquely numbered void tape.
- Seal the styrofoam box at exactly the same place as it was sealed before.
- Proceed carefully and slowly and press the tamper-evident void tape firmly onto the styrofoam box.

WARNINGS AND PRECAUTIONS

- **CAUTION:** Do not use the product if the package is damaged or the seal is broken.
- Discard unused, damaged, or leaking kits in accordance with local, state, and federal regulations.
- **CAUTION:** Do not use incomplete or defective kits.
- **CAUTION:** Do not use the product beyond its expiration date (use-by).
- All items included in the Partial Sample Sealing Kit Version Sydney are NOT provided sterile.

- Storage conditions: between 0°C and + 40°C.

- All items included in the Partial Sample Sealing Kit Version Sydney are single-use items.
- Do not re-use any item.

- Optimal testing performance requires adequate specimen collection and handling. Ensure the outside of the transport packaging and collection caps are not contaminated.
- Take care to avoid cross-contamination during the specimen handling steps. Specimens can contain pathogens. Ensure that containers do not contact one another, and discard used materials without passing over the containers. If gloves come into contact with specimen, change gloves to avoid cross-contamination.
- If the contents are spilled at any time during the collection procedure, use a new collection kit. Failure to use a new kit may invalidate the test results.

ADDITIONAL HELPFUL MATERIALS

For more information on the BEREG-Kit Urine Sample Container kit, consult the following resources:

The latest version of the user manual / handling instructions can be found on our website

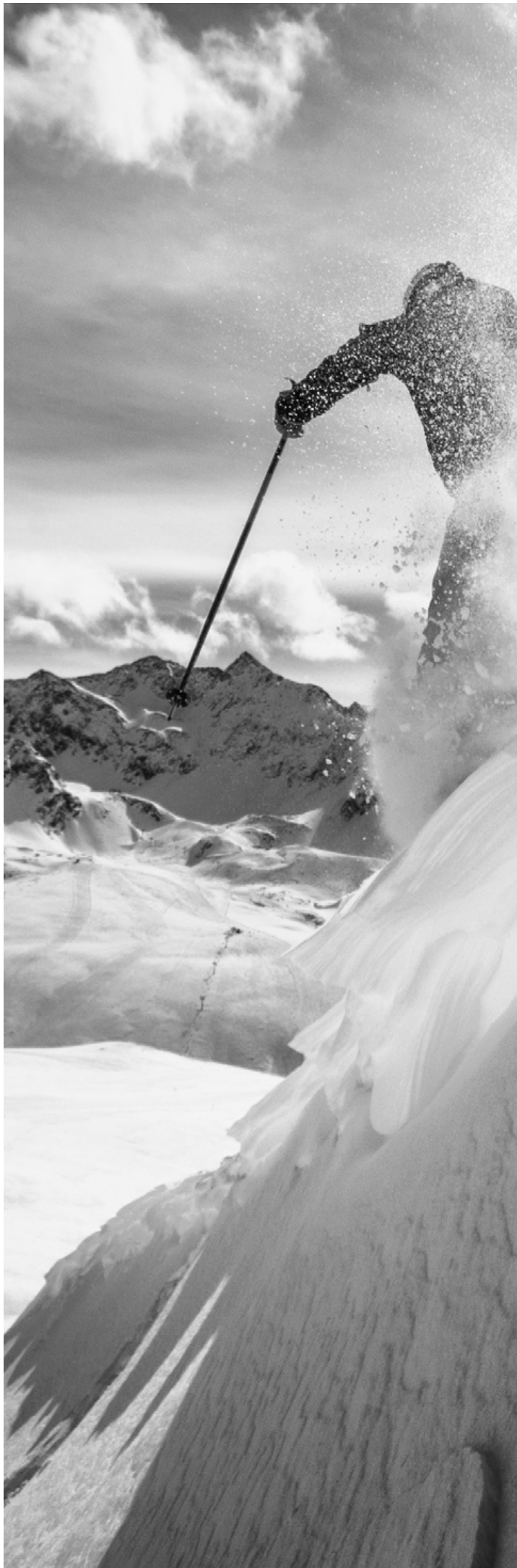
www.berlinger.com/handling-instructions

For additional information, check out the BEREG-Kit product webpage

www.berlinger.com/bereg-kit

For more information, including ways to contact Berlinger, check out our website

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