USER MANUAL

Partial Sample Void

ENGLISH, PAGE	2
DEUTSCH, AB SEITE	6
FRANÇAIS, DE LA PAGE	10
ESPAÑOL, A PARTIR DE LA PÁGINA	14

Berlinger Special AG

Mitteldorfstrasse 2
9608 Ganterschwil
Switzerland

Tel. +41 71 982 88 11
info.special@berlinger.com
www.berlinger.com

TABLE OF CONTENTS

WHAT IS INCLUDED	2
INTENDED USE	2
HANDLING INSTRUCTION	3
WARNINGS AND PRECAUTIONS	4
ADDITIONAL HELPFUL MATERIALS	4

WHAT IS INCLUDED

In this **Partial Sample Void**, you will find:

- One (1) security tape (void) with numbering and barcode

INTENDED USE

Partial Sample Void is intended to be used for temporary sealing of urine partial samples for the purpose of anti-doping and drugs of abuse testing according to WADA ISTI requirements.

This product is not a Medical Device (MD) and therefore is not intended to be used in a medical setting nor on the order of a health care professional. It is also not indicated for or in assistance of disease screening or diagnosis.

Please Note: The manufacturer does in particular not assume any liability for the application of incomplete or defective products or any unlawful actions during the entire doping control procedure.

HANDLING INSTRUCTION

PREPARATION

Please make sure that the content of the Partial Sample Void is complete:

- Security tape (void), with numbering and barcode

HANDLING STANDARDS AND REGULATIONS

This product should be used in accordance with the current International Standard for the Protection of Privacy and Personal Information (ISPPPI) and International Standard for Testing and Investigation (ISTI) established by the World Anti-Doping Agency (WADA).

HANDLING THE PARTIAL SAMPLE VOID

1. When you have a partial sample (less than 90 ml), securely close the cap of the Urine Collection Vessel and check that the foil on the spout is still intact.

Attention: Do not touch the foil on the spout to avoid accidental opening or removal.

The foil must remain intact to ensure that the spout is watertight, which is particularly important for the partial sample sealing process.

2. Remove the backing paper from the security tape (void) and place it over the collection vessel cap covering the spout in a straight line. The athlete must ensure to press the security tape firmly on the cap and on the sides of the Urine Collection Vessel.

According to individual guidelines, the DCO will repeat the same actions, to double-check that the Urine Collection Vessel is securely sealed.

3. Now that the partial sample is temporarily securely sealed, please note down the number from the security tape (void) in accordance with WADA ISTI regulations.

4. Before removing the security tape (void) ensure that no tampering has occurred. To remove the security tape (void), pull off from the opposite side of the spout. The security tape should leave clear trace on the Urine Collection Vessel.


Please note: If the foil from the spout peels off together with the security tape (void), do not be alarmed.

WARNINGS AND PRECAUTIONS

- Do not use the product if the package is damaged or the seal is broken.
- Do not use incomplete or defective equipment or if package is damaged.
- Do not use the product beyond its expiration date (use-by).
- The item included in the Partial Sample Void is NOT provided sterile.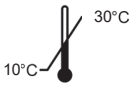

- Storage conditions of the product before use: between +10°C and +30°C

- The Partial Sample Void is a single-use items.
- Do not re-use it.

- Optimal testing performance requires adequate specimen collection and handling. Ensure the outside of the transport packaging is not contaminated.
- Take care to avoid cross-contamination during the specimen handling steps. Specimens can contain pathogens. If gloves come into contact with specimen, change gloves to avoid cross-contamination.

ADDITIONAL HELPFUL MATERIALS

For more information on the product, consult the following resources:

The latest version of the user manual can be found on our website

www.berlinger.com/handling-instructions

For additional information, check out the product webpage

www.berlinger.com/void

For more information, including ways to contact Berlinger, check out our website

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