USER MANUAL

Berlinger Urine Collection Vessel

ENGLISH, PAGE	2
DEUTSCH, AB SEITE	7
FRANÇAIS, DE LA PAGE	12
ESPAÑOL, A PARTIR DE LA PÁGINA	17

Berlinger Special AG

Mitteldorfstrasse 2
9608 Ganterschwil
Switzerland

Tel. +41 71 982 88 11
info.special@berlinger.com
www.berlinger.com

TABLE OF CONTENTS

WHAT IS INCLUDED	2
INTENDED USE	2
HANDLING INSTRUCTION	3
WARNINGS AND PRECAUTIONS	4
ADDITIONAL HELPFUL MATERIALS	4

WHAT IS INCLUDED

In this **Berlinger Urine Collection Vessel**, you will find:

- One (1) collection vessel
 - One (1) green cap
-

INTENDED USE

The Berlinger Urine Collection Vessel is intended to be used to collect urine samples for the purpose of anti-doping and drugs of abuse testing according to WADA ISTI requirements.

This Urine Collection Vessel is not a Medical Device (MD) and therefore is not intended to be used in a medical setting nor on the order of a health care professional. It is also not indicated for or in assistance of disease screening or diagnosis.

Please note: The manufacturer does in particular not assume any liability for the application of incomplete or defective products or any unlawful actions during the entire doping control procedure.

HANDLING INSTRUCTION

PREPARATION

Before opening the packaging, please check if the Urine Collection Vessel is properly sealed or shows any signs of manipulation:

- The plastic packaging is intact.

1. Only open the lower part of the packaging which is marked with „1“ and tear open the perforation. The upper part containing the cap remains closed.
2. Take out the vessel.

Attention: Do not touch the inside of the vessel!

HANDLING STANDARDS AND REGULATIONS

This product should be used in accordance with the current International Standard for the Protection of Privacy and Personal Information (ISPPPI) and International Standard for Testing and Investigation (ISTI) established by the World Anti-Doping Agency (WADA).

FILL LEVEL AND SCALE

The fill level, which is marked by a molded scale, gives physical indication at:

- 90 ml - minimum volume of urine for analysis.
- 150 ml - required to define the parameters needed for accurate gravity measurement.

Please note: The maximum fill level is 180 ml.

Attention: Do not fill above this maximum fill level!

HANDLING THE BERLINGER URINE COLLECTION VESSEL

Collect the urine sample according to WADA ISTI guidelines in the collection vessel.

1. Place the vessel on a firm and even surface. Open the upper part of the packaging which is marked with “2” and take out the green cap.

Attention: Do not touch the inside of the cap!

2. Place the green cap onto the collection vessel and close it by turning the cap in a clockwise direction. This should be continued until the cap can be turned no further.

Attention: Do not touch the foil on the spout to avoid accidental opening or removal.

The foil must remain intact to ensure that the spout is watertight, which is particularly important for the partial sample sealing process.

HANDLING INSTRUCTION

DECANTING THE URINE

1. Before decanting the sample into the Urine Sample Container, ensure to remove the foil from the spout, by pulling it from the middle of the cap towards the outside.
2. For better guidance place the spout on the edge of the opening of the container to decant.

Please note: Do not cover the ventilation hole during decanting.

MEASURING SPECIFIC GRAVITY

Depending on the pipette used to collect a drop of urine for specific gravity measurement, it can be inserted directly through the spout without removing the cap.


This eliminates the risk of spillage when opening a vessel for better access.

WARNINGS AND PRECAUTIONS

- Do not use the product if the package is damaged or the seal is broken.
- Discard unused, damaged, or leaking Berlinger Urine Collection Vessel in accordance with local, state, and federal regulations.
- Do not use incomplete or defective equipment or if package is damaged.
- Do not use the product beyond its expiration date (use-by).
- All items included in the Berlinger Urine Collection Vessel are NOT provided sterile.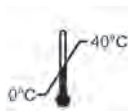

- Storage conditions of the Berlinger Urine Collection Vessel before use: between 0°C and + 40°C.

- The Berlinger Urine Collection Vessel is a single-use item.
- Do not re-use it.

- Optimal testing performance requires adequate specimen collection and handling. Ensure the outside of the transport packaging is not contaminated.
- Take care to avoid cross-contamination during the specimen handling steps. Specimens can contain pathogens. If gloves come into contact with specimen, change gloves to avoid cross-contamination.
- If the contents are spilled at any time during the collection procedure, use a new Berlinger Urine Collection Vessel.

ADDITIONAL HELPFUL MATERIALS

For more information on the product, consult the following resources:

The latest version of the user manual can be found on our website

www.berlinger.com/handling-instructions

For additional information, check out the product webpage

www.berlinger.com/urine-collection-vessel

For more information, including ways to contact Berlinger, check out our website

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