

USER MANUAL

Manual Opener

for

BEREG-Kit

BEREG-Kit Plastic

BEREG-Kit small

BEREG-Horse-Kit



Berlinger Special AG

Mitteldorfstrasse 2
9608 Ganterschwil
Switzerland

Tel. +41 71 982 88 11
info.special@berlinger.com
www.berlinger.com

HANDLING INSTRUCTION

WHAT IS INCLUDED

The opener includes the following items:

1. One (1) manual opener BEREG-Kit
2. One (1) cone for BEREG-Kit / BEREG-Kit Plastic / BEREG-Horse-Kit
3. One (1) cone for BEREG-Kit small (optional)
4. One (1) workholder BEREG-Kit small (optional)
5. One (1) slip-resistant mat



HANDLING THE MANUAL OPENER

TO OPEN THE BEREG-Kit / BEREG-Kit Plastic / BEREG-Horse-Kit



1. Put the metal cone for „BEREG-Kit / BEREG-Kit Plastic / BEREG-Horse-Kit“ on the top of the container.
2. Place the container and the metal cone into the stand of the opener. Please make sure that the neck of the bottle is inserted precisely in the determined slot.
3. Now, use a hammer and hit with reasonable force onto the cone.
4. Open the cap by turning it counterclockwise.




Important: Protect your hands by using the slip-resistant mat for opening the cap.

HANDLING INSTRUCTION

TO OPEN THE BEREG-Kit small



1. Put the metal cone for BEREG-Kit small on the top of the container.
2. Place the workholder BEREG-Kit small on the top of the machine's black stand until it clicks into place.
3. Place the container and the metal cone into the stand of the opener. Please make sure that the neck of the bottle is inserted precisely in the determined slot.
4. Now, use a hammer and hit with reasonable force onto the cone.
5. Open the cap by turning it counterclockwise.

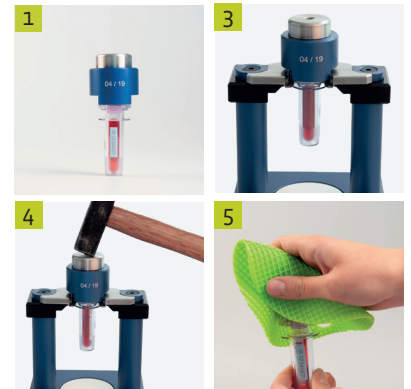

Important: Protect your hands by using the slip-resistant mat for opening the cap.

WARNING

- Use eye protection.



For more information on the product, consult the following resources:

The latest version of the user manual can be found on our website

www.berlinger.com/ado-user-manuals

For additional information, check out the product page

www.berlinger.com/doping-control/accessory/manual-opener

For more information, including ways to contact Berlinger, check out our website

www.berlinger.com



berlinger
feel safe