

Mini-KIT

The Mini-KIT was developed in order to enable an easy process for blood screening tests on site.


This short "screening" test is designed to find out whether or not a further urine or blood test has to be conducted and analysed. Usually these tests are taken just before the competition. To keep the procedure as simple as possible and to earn the athletes confidence, the Mini-KIT contains all necessary utensils.

The Mini-KIT is designed to be used for screening tests on site - NOT to be sent to the laboratory.

One kit contains:

- 1 BD Vacutainer EDTA 3ml (PET)
- 1 BD security needle 21G
- 1 numbered security void tape "Blood-KIT"
- 4 small labels with number and barcode

The Mini-KIT is uniquely numbered on the box, the 4 small labels as well as on the security void tape.

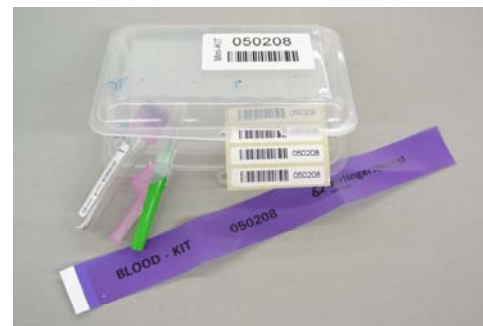

Handling information Mini-KIT

1. Before you take off the security void tape of the Mini-KIT please check that it is still untouched.
2. After opening the kit please ensure that all numbered components have the same number.
3. Use the Mini-KIT according to your guidelines and instructions. (The small numbered labels can be stuck onto the tubes or documents)
4. After the blood collection put the tube back into the plastic box.
5. Reseal it with the numbered security void tape. Important for sealing:

Please make sure that for the resealing the numbered security void tape is placed at the same position as the first security tape has been placed.

