

Partial Sample Version Sydney

If an athlete cannot initially provide the required amount of urine, the partial sample needs to be temporarily securely sealed (in accordance with the International Standard for Testing, WADA IST).

The Partial Sample Version Sydney must only be used for BREG-KIT's in a styrofoam box.

Contents

- 1 uniquely numbered tamper-evident void tape
- 1 white plastic stopper
- 1 handling information sheet



Handling information

- Fill the collected urine into the B-bottle and close the partially filled bottle with the white plastic stopper.
Leave the red ring on the bottle neck!
The A-bottle remains unopened.
- Place the numbered security cap back on top of the B-bottle.
Do not screw the cap.
- Put all components back into the styrofoam box and close it
- Seal the styrofoam box with the uniquely numbered tamper-evident void tape. The BREG-KIT is now temporarily sealed.



Recommendations for an easy handling

- Use the white strap to remove the backing paper from the uniquely numbered void tape.
- Seal the Styrofoam box at exactly the same place as it was sealed before.
- Proceed carefully and slowly and press the tamper-evident void tape firmly onto the styrofoam box.



The manufacturer shall not be held liable if the Partial Sample Version Sydney was used beyond the manufacturers given limitations and handling instructions. Please note that any instructions for use published prior to this dated guide are no longer valid!