

## Manual compressor

Metal cap for the BREG-KIT „urine“

Metal cap for the BREG-KIT small

### Handling instruction

1. Put the glass bottle or the plastic container into the plastic cup.  
(white cup for glass bottles, grey cup for plastic containers)
2. Cover the security cap with the metal cap.
3. Place it under the Manual compressor.
4. Turn the wheel of the compressor until you can hear a cracking noise and it stops. After that turn the wheel back again.
5. Remove the metal break cap and turn off the separated security cap.  
(to protect your hands from injuries, use the plastic pad to open the cap)



### Important

Machine must be fixed with manual screw thread!

If you cannot fix the machine with screw thread please use the added "velcro" pads.  
(use the loop-part on the table and the hook-part on the machine)



### Recommendation

Use once a month a drop of oil

