

Partial Sample Version Bavaria

If an athlete cannot provide the required amount of urine, the partial sample needs to be temporarily and securely sealed. (In accordance with the International Standard for Testing, WADA IST)

Contents

- 1 numbered security plastic bag
- 1 white plastic stopper
- 1 handling information sheet




Handling information

- Fill the collected urine into the B-bottle.
Close the partially filled B-bottle with the white plastic stopper.

Leave the red ring on the bottle neck.
The A-bottle remains unopened.

- Place the numbered security cap back on top of the B-bottle and do not screw the cap.
- Put all components back into the styrofoam / cardboard box and close it.
- Place all components into the security plastic bag, so that the bottles are in an upright position!
- Seal the security plastic bag by removing the backing paper from the tamper evident void tape and press it firmly together.
- The BEREG-KIT is now temporarily securely sealed.

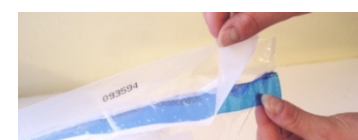

The manufacturer shall not be held liable if the Partial sample Version Bavaria was used beyond the manufacturers given limitations and handling instructions.
Please note that any instructions for use published prior to this dated guide are no longer valid!