USER MANUAL

Partial Sample Bag

ENGLISH, PAGE	2
DEUTSCH, AB SEITE	6
FRANÇAIS, DE LA PAGE	10
ESPAÑOL, A PARTIR DE LA PÁGINA	14

Berlinger Special AG

Mitteldorfstrasse 2
9608 Ganterschwil
Switzerland

Tel. +41 71 511 87 11
info.special@berlinger.ch
www.berlinger.ch

TABLE OF CONTENTS

WHAT IS INCLUDED	2
INTENDED USE	2
HANDLING INSTRUCTION	3
WARNINGS AND PRECAUTIONS	4
ADDITIONAL HELPFUL MATERIALS	4

WHAT IS INCLUDED

In this **Partial Sample Bag**, you will find:

- One (1) security bag made of recycled plastic with numbering and barcode

INTENDED USE

Partial Sample Bag is intended to be used for temporary sealing of urine partial samples for the purpose of anti-doping and drugs of abuse testing according to WADA ISTI requirements.

This product is not a Medical Device (MD) and therefore is not intended to be used in a medical setting nor on the order of a health care professional. It is also not indicated for or in assistance of disease screening or diagnosis.

Please note: The manufacturer does in particular not assume any liability for the application of incomplete or defective products or any unlawful actions during the entire doping control procedure.

HANDLING INSTRUCTIONS

PREPARATION

Please make sure that the content of the Partial Sample Bag is complete:

- Security bag made of recycled plastic with numbering and barcode

HANDLING STANDARDS AND REGULATIONS

This product should be used in accordance with the current International Standard for the Protection of Privacy and Personal Information (ISPPPI) and International Standard for Testing and Investigation (ISTI) established by the World Anti-Doping Agency (WADA).

HANDLING THE PARTIAL SAMPLE BAG

1. When you have a partial sample (less than 90 ml), securely close the cap of the Urine Collection Vessel and check that the foil on the spout is still intact.

Attention: Do not touch the foil on the spout to avoid accidental opening or removal. The foil must remain intact to ensure that the spout is watertight, which is particularly important for the partial sample sealing process.

2. Place the Urine Collection Vessel in the bag and close it by removing the backing paper from the adhesive tape and press it firmly together.

According to individual guidelines, the DCO will double-check that the Partial Sample Bag is securely closed.

3. Now that the partial sample is temporarily securely sealed, please note down the number from the bag in accordance with WADA ISTI regulations.

4. Before opening the bag, ensure that no tampering has occurred. To open the security bag, tear at perforated area which is located on each upper side of the bag.

Attention: The perforated areas on the upper side of the bag do not allow for any tampering!


WARNINGS AND PRECAUTIONS

- Do not use the product if the package is damaged or the seal is broken.
- Do not use incomplete or defective equipment or if package is damaged.
- Do not use the product beyond its expiration date (use-by).

- The item included in the Partial Sample Bag is NOT provided sterile.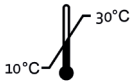

- Storage conditions of the product before use: between +10°C and +30°C.

- The Partial Sample Bag is a single-use item.
- Do not re-use it.

- Optimal testing performance requires adequate specimen collection and handling. Ensure the outside of the transport packaging is not contaminated.
- Take care to avoid cross-contamination during the specimen handling steps. Specimens can contain pathogens. If gloves come into contact with specimen, change gloves to avoid cross-contamination.

ADDITIONAL HELPFUL MATERIALS

For more information on the product, consult the following resources:

The latest version of the user manual can be found on our website

www.berlinger.ch/handling-instructions

For additional information, check out the product webpage

www.berlinger.ch/partial-sample-bag

For more information, including ways to contact Berlinger, check out our website

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